

**EN INFORMATION**  
**Irisprint contact lenses**



**Name and address of manufacturer**  
 Falco Linsen AG  
 Talackerstrasse 14  
 CH-8274 Tägerwilen  
 +41 71 666 70 40  
 Switzerland



All rights reserved. No part of this document may be reproduced, transmitted, transferred, distributed, altered, summarised, or translated into another language in any form or by any means, whether graphical, electronic, or mechanical, and in particular through photocopying, recording, or data storage and processing within systems without the prior written permission of the manufacturer. Copying the system software is not permitted.

Copyright © 2020 Falco Linsen AG. All rights reserved.



**IMPORTANT:** Please read through these instructions for use completely before using any of the products.

**Contents**

- 1 Intended purpose
- 2 Safety precautions for use
- 3 Additional information

**Symbols used in this document:**

Symbol:	Erklärung:
	Symbol for “Consult instructions for use”
	Symbol for “Caution”

The symbols are taken from EN ISO 15223-1:2017.



Your contact lens contains an individual, hand-finished iris print. Please note the following points in addition to the enclosed instructions for use.

# 1 Intended purpose

## Indications:

Covering of defective sections of the globe of the eye

- For the iris in the case of iris coloboma, congenital or traumatic mydriasis, albinism
- Iris lens for cloudy cornea

Cosmetic indications:

- Concealing an iris disfigured by accident or disease

# 2 Safety precautions for use

- Before you drive while wearing your iris print contact lens, the practitioner (qualified optician, optometrist, or ophthalmologist) must check your fitness to drive while wearing it (pinhole aperture effect).
- If you experience a foreign body sensation, irritation, redness of the eye and/or excessive tearing, please refer to the note in Section 7.1 of the instructions for use.
- Do not use intensive cleaning agents, as this may cause the iris print to become dull.

# 3 Additional information

Every iris print is unique. Because each product is finished individually by hand, there may be slight differences in structure and colouring.

For a larger print, we recommend the digital version of this document which can be found on our website.

Document version: 2020-02-15, version 3