

CONTACT DISTANCE VALUE
CUSTOMISED MADE TO MEASURE ENJOYMENT TRUST
QUALITY COMFORT
RIGID GAS PERMEABLE
CONTACT LENSES
RANGE VITALITY
BRILLIANCE UNDERSTANDING FLEXIBILITY INDIVIDUALITY SENSE
LIFE UNIQUE ENJOYMENT WISHES FRIENDS
DISTANCE

HANDLING OF
RIGID GAS PERMEABLE
CONTACT LENSES

falco-linsen.com

falco

APPLICATION

- For the correction of ametropia and visual optimisation of the healthy eye.

GENERAL

- Always wash and dry your hands before touching your contact lenses or eyes.

CONTRADICTIONS

- Inflammations, infections and irritation of the eye.
- Change in corneal sensitivity, poor tear quality.
- Systemic diseases which are linked to a weakened immune system and eyelid dysfunction, please consult your eye care practitioner.

INTERACTIONS WITH OTHER SUBSTANCES

- To prevent unwanted interactions with the contact lenses and care products please use only the care products recommended by your eye care practitioner.
- Pharmaceuticals can impact the tear film and may affect the tolerance and comfort to wear contact lenses.

- Medications can impact the tear film and may affect your tolerance and ability to wear contact lenses.
- Please inform your eye care practitioner of any medications you are using.

SIDE EFFECTS

- Any changes in wearing comfort, such as excessive tear flow, redness of the eye, loss of vision, or allergic reaction, should be reported to your eye care practitioner. Contact lenses should be removed immediately until your eye care practitioner has been consulted.

SPECIAL INSTRUCTIONS

- Your eye care practitioner should be consulted on a regular basis to assess your lens(es).
- Your contact lenses must always be carefully cleaned and maintained in line with the respective instructions.
- Do not rinse your contact lens or its container with tap water.
- The recommended wearing time for your contact lenses should not be exceeded.
- Do not wear your contact lenses during sleeping unless your eye care practitioner advised you to.
- In the case of an eye irritation, the contact lenses are to be removed immediately and your eye care practitioner should be consulted.
- Do not use damaged or broken contact lenses.

V1.0

falco